ESOCLEAN RGP

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ESOCLEAN RGP

DETERGENT FOR RIGID GASPERMEABLE
CONTACT LENSES

PROPERTIES: Contact lenses require periodically a deep and accurate cleaning. Surfactants molecules clean deeply granting an excellent comfort and an improved biotolerability. **INDICATIONS:** It's suitable for daily cleaning of all types of rigid gaspermeable contact lenses.

INSTRUCTIONS FOR USE: Wash hands accurately with a clean cloth. - To avoid mistakes, start the treatment always with the same lens. - Remove the lens from the eye and put it on the hand palm adding a quantity of ESOCLEAN RGP, enough to wet the lens completely. - Rub softly with "backwards and forwards" movements, both lens surfaces for 10-15 seconds, using a finger of the other hand. - Rinse carefully and abundantly the lens with a multipurpose or saline solution to remove all deposits and traces of product. - Proceed to disinfection. - Repeat the same operations with the other lens.

CONTRAINDICATIONS: There are no ascertained contraindications, apart from allergy or specific hypersensitivity to some component.

PRECAUTIONS: The product, once open, must be used within three months. The left liquid must be disposed. Do not use in case of damaged seal. Ensure that the bottle is well closed after product use. Store the bottle away from heat fonts and in any case under 30°C.

Do not use after the expiry date:

WARNINGS: Do not ever apply the lens without previously rinsing it with the appropriate rinsing solutions: the solution must never come into contact with the eye. In case of accidental contact wash the eye immediately with plenty of water. Do not use in case of specific hypersensitivity to some component. Keep it far from children reach. The product is for external use only. Do not replace with a different detergent. Seek professional advice before using any ophthalmic drug while wearing contact lenses. **COMPLICATIONS:** In case of persistent trouble or irritation, discontinue use and consult a specialist. **PRODUCT VALIDITY:** 3 years, if the product is kept in integral package and correctly preserved.

Follow the instructions on the label strictly and keep the present technical-informative note.

COMPOSITION: Polyhexamethylene Biguanide 0.00005%; tetrasodium edetate 0.2%; Surfactants 6.0%; made up to 100% with Excipients and Purified water.

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