**esoform****ESOSTILL - GOCCE OCULARI**CON ACIDO IALURONICO 0,2%
E ACQUE DISTILLATE DI FIORI**ESOSTILL - EYE DROPS**WITH HYALURONIC ACID 0.2%
AND FLOWER DISTILLED WATERS

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PROPERTIES: The product contains hyaluronic acid, Hamamelis distilled water, Chamomile distilled water, Bluebottle distilled water, Euphrasy distilled water.

Hyaluronic acid is a highly tolerated natural polymer, which is present both in lacrimal film and in human eye and has the property of soothing irritation of eye surface, while maintaining it optimally moistened. Hamamelis, Chamomile and Euphrasy distilled waters boost the solution moistening properties and have a refreshing and lenitive effect, whereas Bluebottle distilled water grants a soothing, anti-redness effect, thanks to the washing away mechanical action. ESOSTILL helps to maintain eye water balance. Its use gives an immediate feeling of relief, freshness and comfort.

MANUFACTURED IN ASEPSIS **STERILE A**

USE INDICATIONS: ESOSTILL eye drops are indicated to relieve eye redness, irritation, fatigue, itch and dryness due to: air conditioning and heating, stay in smoky places or with other kinds of air pollution, prolonged exposure to wind or sun rays, chlorine in case of swimming practice in swimming pools, or saltiness if such activity is practiced in the sea, sport activities in general, deep concentration in study or work activities, use of contact lenses, post operating traumas, external bodies, excessive use of screen terminals and tiredness.

USE INSTRUCTIONS: Put one or two drops in each eye squeezing the bottle gently. The product can be used every day, more times a day, according to necessities, also while wearing contact lenses. ESOSTILL use is simple, safe and without side effects; it can be used by everyone, including children and elderly. ESOSTILL was tested to guarantee the absence of cytotoxicity in case of direct contact (UNI EN ISO 10993.5) and to exclude irritation (UNI EN ISO 10993.10).

CONTRAINDICATIONS: There are no ascertained contraindications, apart from specific allergy to some component.

PRECAUTIONS: The product is for external use only. The solution, once opened, must be used within 90 days. Do not use in case of damaged seal. Never put the dropper in contact with any kind of surface, including eye one, not to contaminate the product. Ensure the bottle is well closed after use. Store the bottle away from heat sources and in case under 30°C. Do not use after the expiry date printed on the packing.

WARNINGS: Do not use in case of specific hypersensitivity to some component. Keep far from children reach. In case of existing therapy with any ophthalmic drug seek professional advice before using ESOSTILL contemporarily. Do not waste the bottle in the environment after use. **CONTRAINDICATIONS:** There are no ascertained contraindications apart from allergy to specific components. **COMPLICATIONS:** In case of trouble or persistent irritation, discontinue use and consult a specialist. **VALIDITY:** 3 years if intact and properly preserved.

Follow the instructions on the label strictly and keep this technical-informative note.

COMPOSITION: PoliEsaMetilenBiguanide: 0.000246%; Sodium Hyaluronate 0.2%; Distilled Waters of Chamomile, Hamamelis, Euphrasy and Bluebottle 40%, Disodium edetate, Borate buffer, Sodium Chloride, Excipients and purified Water up to 100%.

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