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FOR THE DISINFECTATION PRESERVATION CLEANING, RINSING, HYDRATION OF ALL
SOFT CONTACT LENSES - REMOVES PROTEINS

Recommended for sensitive eyes - Active in 5-7 minutes - New multipurpose with sodium hyaluronate.

PROPERTIES: The multipurpose ESOLAL has a highly disinfecting preserving, wetting and cleaning power. NEW!!!! With Sodium Hyaluronate. This polymer with highly visco-elastic properties improves the lubrication and bio-tolerance of the lenses thanks to its water-lock property. ESOLAL hydrates the lenses both immediately after their application and during day wear with consequent prolonged comfort and reduction of eye dryness and irritation. The PH and osmolarity have been purposively studied to be perfectly bio-compatible with ocular tissues and organic liquids.

MANUFACTURED IN ASEPSIS **STERILE A**

INDICATIONS: It's suitable for preservation, disinfection, rinse, cleaning, hydration, daily proteins removal of all kinds of soft contact lenses.

INSTRUCTIONS FOR USE

METHOD 5-7 MINUTES

- > Wash hands and dry accurately with a clean cloth.
- > To avoid mistakes, start the treatment always with the same lens.
- > Remove the lens from the eye and put it on the hand palm adding a quantity of ESOLAL, enough to wet the lens completely.
- > Rub softly with "backwards and forwards" movements, both lens surfaces for 10-15 seconds, using a finger of the other hand.
- > Rinse carefully and abundantly the lens with ESOLAL in order to remove all deposits.
- > Put the lens in the container provided.
- > Repeat the same operations with the other lens.
- > Fill the container with ESOLAL until the dosage line.
- > Close the container accurately and leave the lenses immersed for at least 5-7 minutes.
- > Before wearing the lenses, rinse them with ESOLAL.

METHOD "DON'T TOUCH"

- > Wash hands and dry them accurately with a clean cloth.
- > To avoid mistake, start the treatment always with the same lens.
- > After use, remove the lens from the eye and place it immediately into the container provided filled with ESOLAL until the dosage line.
- > Repeat the same operation with the other lens.
- > Close the container accurately and leave the lenses immersed for at least 2-3 hours or all night.
- > Before re-wearing, rinse them with ESOLAL.

CONTRAINDICATIONS: There are no ascertained contraindications, apart from allergy or specific hypersensitivity to some component.

PRECAUTIONS: ESOLAL once open, must be used within three months. The left liquid must be disposed. Do not use in case of damaged seal, as sterility cannot be guaranteed. To avoid any contamination, do not put the dropper in contact with any kind of surface. Ensure that the bottle is well closed after product use. Store the bottle away from heat fonts and in any case under 30°C.

Do not use after the expiry date:

WARNINGS: For lenses remained in the ESOLAL solution over 3-4 weeks, before use eliminate the old solution from the container and proceed with disinfection, at least according method 1 (5 - 7 minutes). Do not use in case of specific hypersensitivity to some component. Keep it far from children reach. Use always fresh solution getting rid of the one remained in the lens-keeping container. Do not replace with a different disinfectant. Seek professional advice before using any ophthalmic drug while wearing contact lenses. Never use tap water for soft contact lenses.

COMPLICATIONS: In case of persistent trouble or irritation, discontinue use and consult a specialist.

VALIDITY: 3 years if intact and properly preserved.

Follow the instructions on the label strictly and keep this technical-informative note.

COMPOSITION: Polyhexamethylene Biguanide 0.0001%; Disodium edetate 0,1%; Sodium Hyaluronate 0.01%; made up to 100% with Excipients and Purified water.

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