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INDICATIONS: It is suitable as lubricating and hydrating solution for both soft and rigid gaspermeable contact lenses.

INSTRUCTIONS FOR USE

WASH HANDS ACCURATELY.

> Take the lens, already disinfected with the Multipurpose, from its special container.

> Put 2-3 drops of ESODROP on the contact lens before applying it.

> Apply the lens.

> The lens can be lubricated also after its application, putting one or two drops.

CONTRAINDICATIONS: There are no ascertained contraindications, apart from specific allergy to some component.

PRECAUTIONS: ESODROP, once open, must be used within three months. Do not use in case of damaged seal. Do not put the dropper in contact with any kind of surface. Ensure that the bottle is well closed after product use. Store the bottle away from heat fonts and in any case under 30°C

Do not use after the expiry date:

WARNINGS: Do not use in case of specific hypersensitivity to some component. **Keep far from children reach.** Do not replace with a different wetting product. Seek professional advice before using any ophthalmic drug while wearing contact lenses. **COMPLICATIONS:** In case of trouble or persistent irritation, discontinue use and consult a specialist.

VALIDITY: 3 years if intact and properly preserved.

Follow the instructions on the label strictly and keep this technical-informative note.

COMPOSITION: Polyhexamethylene Biguanide 0.00023%, Sodium hyaluronate macromolecular 0.2%; Disodium edetate 0.1%; Surfactants 0.0015%; made up to 100% with Excipients and purified water.

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