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**ESOSAN RGP**  
DISINFETTANT FOR RIGID GASPERMEABLE  
CONTACT LENSES

**PROPERTIES:** The Solution ESOSAN RGP, thanks to the synergy between its ingredients and innovative components, offers a highly disinfecting, preserving and protective action. It's an isotonic and isohydric solution for the eyeball cells; it is therefore biotolerable and suitable for very sensitive eyes, too. The effective detergent power makes the lens very hydrated both after immediate application and during day use, with a consequent higher comfort.

MANUFACTURED IN ASEPSIS **STERILE A**

**INDICATIONS:** It's suitable for preservation, disinfection, rinse, cleaning of all kinds of rigid gaspermeable contact lenses.

**INSTRUCTIONS FOR USE**

**1 - METHOD 8-10 MINUTES**

- > Wash hands and dry them accurately with a clean cloth.
- > To avoid mistakes, start the treatment always with the same lens.
- > Remove the lens from the eye and put it on the hand palm adding a quantity of ESOSAN RGP, enough to wet the lens completely.
- > Rub softly with "backwards and forwards" movements, both lens surfaces for 15-20 seconds, using a finger of the other hand.
- > Rinse carefully and abundantly the lens with ESOSAN RGP to remove all deposits.
- > Put the lens in the container provided.
- > Repeat the same operations with the other lens.
- > Fill the container with ESOSAN RGP until the dosage line.
- > Close the container accurately and leave the lenses immersed for at least 8-10 minutes.

> Before wearing the lenses, rinse them with ESOSAN RGP.

**2 - METHOD "DON'T TOUCH"**

- > Wash hands and dry them accurately with a clean cloth.
- > To avoid mistake, start the treatment always with the same lens.
- > After use, remove the lens from the eye and place it immediately into the container provided filled with ESOSAN RGP until the dosage line.
- > Repeat the same operation with the other lens.
- > Close the container accurately and leave the lenses immersed for at least 3-4 hours or all night.
- > Before re-wearing, rinse them with ESOSAN RGP.

**3 - METHOD "DEEP CLEANING"**

- > Wash hands and dry them accurately with a clean cloth.
- > To avoid mistakes, start the treatment always with the same lens.
- > Remove the lens from the eye and put it on the hand palm adding a quantity of ESOSAN RGP, enough to wet the lens completely.
- > Rub softly with "backwards and forwards" movements, both lens surfaces for 15-20 seconds, using a finger of the other hand.
- > Rinse carefully and abundantly the lens with ESOSAN RGP to remove all deposits.
- > Fill the container with ESOSAN RGP until the dosage line.
- > Close the container accurately and leave the lenses immersed for at least 3-4 hours or all night.
- > Before wearing the lenses, rinse them with ESOSAN RGP.

**CONTRAINDICATIONS:** There are no ascertained contraindications, apart from allergy or specific hypersensitivity to some components. **PRECAUTIONS:** ESOSAN RGP, once open, must be used within three months. The left liquid must be disposed. Do not use in case of damaged seal, as sterily cannot be guaranteed. To avoid any contamination, do not put the dropper in contact with any kind of surface. Ensure that the bottle is well closed after product use. Store the bottle away from heat fonts and in any case under 30°C.

**Do not use after the expiry date:**

**WARNINGS:** For lenses remained in the ESOSAN RGP solution over 3-4 weeks, before use eliminate the old solution from the container and proceed with disinfection, at least according method 1 (8-10 minutes). Do not use in case of specific hypersensitivity to some component. Keep it far from children reach. Use always fresh solution getting rid of the one remained in the lens-keeping container. Do not replace with a different disinfectant. Seek professional advice before using any ophthalmic drug while wearing contact lenses. **COMPLICATIONS:** In case of persistent trouble or irritation, discontinue use and consult a specialist. **VALIDITY:** 3 years if intact and properly preserved.

**Follow the instructions on the label strictly and keep this technical-informative note.**

**COMPOSITION:** PolyhexaMethylene Biguanide: 0.00005%; Disodium edate: 0.1%; Ampho-cationic Surfactants 0.402% made up to 100% with Excipients and Purified Water.

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